

TRACK & TRAIL



Available exclusively through

FORTS Technical Solutions

Brad Fortune

Mob: 0419 959 157

Email: fortstech@iinet.net.au

Web: www.fortstechnicalsolutions.com.au

 Like us on
Facebook Forts Technical Solutions



ASSEMBLY, OPERATION AND SAFETY MANUAL

1. Operating the wheelchair

The Track & Trail wheelchair is designed for general outdoor use in parks, beaches and bush tracks.

It is not intended to be self-propelled.

Do not leave children in the chair unattended.

Do not attempt steep declines.

The wheelchair is not intended for extreme conditions or to be submersed.

For safety purposes please avoid using the wheelchair in adverse or poor conditions.

1.1 User indications

The wheelchair is intended to have an attendant present for operating purposes at all times.

Operating the wheelchair may injure the user or damage the wheelchair if the user is unable to have full control.

2. Safety and Operating Information

This manual refers to standard operating procedures in normal outdoor conditions.

Use in extreme conditions may void warranty.

Please adhere to safe operation and maintenance of the wheelchair to ensure best performance.

In order to handle your wheelchair safely please follow the instructions in the manual.

Stability

The wheelchair has been designed to provide stability during normal daily activities therefore with proper operation the wheelchair will maintain balance at all times.

Any changes or modifications to the wheelchair unauthorised by the manufacturer may affect the centre of gravity and may lead to the wheelchair tipping over and cause injury.

Weight Limit

The maximum weight limitation of the user is 130kg.

CAUTION!

Do not use parts that can be detached, such as armrests, for support or any other reason other than the intended purposes. Do not use them as a means for lifting the product as they may be inadvertently released. This may cause injury.

3. Safety Inspection

3.1 Performance

As the operator, any possible operational defects of your wheelchair will be noticed by you first. The easiest troubleshooting symptoms to identify and the preliminary inspection that you perform are indicated in the following table.

You should consult the manufacturer if the symptoms persist after adjusting the pressure in the tyres and tightening screws and nuts.

3.2 General Inspection

The responsibility of any wheelchair repairs belongs to the manufacturer with qualified, technical expertise.

Regular maintenance helps in the identification and replacement of worn parts on the wheelchair.

This also ensures best performance of the wheelchair.

We recommend a complete inspection by your manufacturer at least once a year.

A. Armrests

Make sure the armrests are firmly attached and in good condition.

B. Seat and Back Upholstery

Make sure that the upholstery is in good condition.

C. Fork

Make sure that all the fittings are well tightened.

D. Rubber pneumatic tyres

Check the pressure of the pneumatics tyres (value indicated on the side).

E. Push handlebar

Ensure handlebar is fitted and secured with pins before use.

*Please contact manufacturer if you have any concerns with parts or performance


4. Assembling the Chair

4.1 Front Wheel Installation

Place chair face up



Locate front wheel section into frame

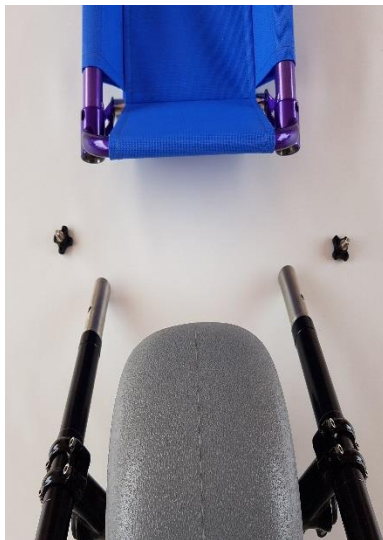

Secure and tighten tri wheel



4.2 Rear Wheel Installation

Align axle with locator tube and depress button to slide into place



Once fully inserted and ensure security button is locked in place



4.3 Backrest Installation

Align and insert side frame of backrest into front locators on chair base



Feed Velcro tab around crossbar and secure at rear



4.4 Push Handlebar Installation

Align and insert pushbar into rear locators on chair base



Secure and tightened tri wheel



4.5 Armrest Installation

Align and slide into and through locator



Insert spring locking pin



****** Ensure all pins, clips and tri wheels are secured**

Your Track & Trail Wheelchair is ready to use



USER MODIFICATIONS

CAUTION:

Forts Technical Solutions cannot accept responsibility for any modifications or alterations made to our components or products after they leave our premises.

Customers modifying or altering our components or products, or using them in conjunction with products from other manufacturers, do so at their own risk since we are unable to validate the safety and efficiency of the modification or altered product.

The modification or alteration of our components or products in conjunction with products from other manufacturers may make the components and products unsafe which could cause **SERIOUS INJURIES.**

Track & Trail wheelchair

WARRANTY

Conditions:

With regards to the original purchaser/user only. Forts Technical Solutions warrants the frames to be free from defects in materials and workmanship for a period of (1) year from date of purchase all remaining components for (1) year from date of purchase except: upholstered materials, padded materials and tyres/wheels.

This warranty does not include the labour or shipping charges incurred for replacement, part installation or repair of any such product.

Forts Technical Solutions obligations and your exclusive remedy under this warranty shall be limited to such repair/replacement.

For warranty service please contact Forts Technical Solutions with your date of purchase, indicate the nature of the defect and the serial number of the product. Do not return our product for warranty service without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED. PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, BUT NOT LIMITED TO, MODIFICATIONS THROUGH THE USE OF UNAUTHORISED PARTS OR ATTACHMENTS; PRODUCT DAMAGE BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF FORTS TECHNICAL SOLUTIONS, OR A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND FORTS TECHNICAL SOLUTIONS CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY FORTS TECHNICAL SOLUTIONS. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS.

Date of purchase: _____

Serial No.: _____